SELECT

PROFCARE

PROFCARE STRETCH EXTRA BANDAGE

Style no. 720062 / 720063

Material:

Fabric: 95% Cotton, 5% Elastane
Glue: 100% latex Adhesive

LATEX Caution: This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.

Medical Device Class I, Rule 1, for non-sterile and noninvasive devices for short term use, by the European Medical Device Regulatory 2017/745.

Basic UDI-DI: 5703543U1004L4

Made in China

For further information and inquiries regarding this product, please contact SELECT SPORT A/S

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EN

Intended use:

The PROFCARE Stretch Extra Bandage is intended to be used for compression and support of muscles and joints.

The bandage is self-adhesive, sticks to itself and does not stick to skin.

It is for use on the skin only.

It is for single, short-term use and is a non-sterile device.

Duration of use should not exceed 3-4 hours

Warnings and contraindications:

Do not use the bandage if you have allergies or are sensitive to some of the ingredients

Contraindications:

Use of the product is contraindicated when:

- the skin is broken
- do not stick on the wound directly
- the duration of use should not exceed 3-4 hours
- skin needs to be cleaned before use.

Warnings:

Do not apply too tight.

Do not use if the user has known allergies to rubber.

Do not use the tape if the packaging is damaged or affected with damp and mildew.

Consult your Health Care Provider if you experience any reaction

Do not store above 25 degrees Celsius. Store in original packaging to protect from light.

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