SELECT PROFCARE

PROFCARE PRO STRAP TAPE

Style no. 720105 / 720106 / 720112 / 720113

Material:

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EN

Intended use:

The PROFCARE Pro Strap tape is intended to be used to support the body parts of sports athletes.
It is intended to prevent injury or reinjury by stabilizing body parts of sports athletes.
To prevent and treat injuries by immobilization. This is a non-elastic tape.
Easy to tear by hand length- and widthwise.
The tape is latex free.
It is for use on the skin only.
It is for single, short-term use and is a non-sterile device.

Instructions for use:

Fix the top of the tape. Keep a constant traction for the first circle. Overlap the next circle with half width on the first circle. To assure good stickiness, let the final circle fully cover the previous circle. Keep a constant pressure with the palm for a few second on the end of the tape.
Skin needs to be cleaned before use.
Duration of use should not exceed 3-4 hours.

Warnings and contraindications:

Contraindications:

Use of the product is contraindicated when:

- the skin is broken
- there is swelling at the application site
- the user has known allergies to rubber
- the product contains Rosin

Warnings:

Do not apply too tight.
Ensure that the skin is cleaned before application

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