SELECT PROFCARE

PROFCARE ELASTIC BANDAGE

Style no. 720092 / 720093

Material:

Fabric: 70% Wrap knitted DTY, 30% Natural rubber
Steel clipse: 78% Aluminium, 12% rubber, 10% Polyester

Caution: This Product Contains
Natural Rubber Latex Which May
Cause Allergic Reactions.

Medical Device Class I, Rule 1, for non-sterile and noninvasive devices for short term use, by the European Medical Device Regulative 2017/745.

Basic UDI-DI: 5703543U1014LB

For further information and inquiries regarding this product, please contact SELECT SPORT A/S

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EN

Intended use:

The PROFCARE Elastic Bandage is intended to be used as support and relief for ankle sprains and distortions. Compression and support of muscles and joints. It is for use on the skin only. Duration of use should not exceed 3-4 hours. PROFCARE Elastic Bandage consists of a beige wrap knit elastic fabric, which can be stretched approx. 180%. The bandages can be washed at 40 degrees Celsius. Comes with 2 fastening clips. After repeated use, the binding loses some of its elasticity and supportive effect. When washed, the binding becomes resilient again. Do not wear the bandage at night.

Warnings and contraindications:

Do not use the bandage if you have allergies or are sensitive to some of the ingredients

Contraindications:

Use of the product is contraindicated when:

- the skin is broken skin
- Do not stick on the wound directly
- the user Duration of use should not exceed 3-4 hours

Warnings:

Do not apply too tight
Do not use if the user has known allergies to rubber
Do not use the tape if the packaging is damaged or affected with damp and mildew.
Consult your Health Care Provider if you experience any reaction
Do not store above 25 degrees Celsius. Store in original packaging to protect from light.

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