# SELECT PROFCARE

## PROFCARE STRETCH TAPE

Style no. 720060 / 720061 / 720097 / 720098

### Material:

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EN

### Intended use:

The PROFCARE Stretch tape is intended to be used to support the body parts of sports athletes.  
This product is intended to be used as a secondary bandage for sports athletes to keep the primary bandage securely in place with sporting injuries.  
The tape is adhesive to the skin and offers elastic support.  
It is for use on the skin only.  
It is for single, short-term use and is a non-sterile device.

### Instructions for use:

Skin needs to be cleaned before use.  
Duration of use should not exceed 3-4 hours.  
The tape is latex free.

### Warnings and contraindications:

Do not use the tape if you have allergies or are sensitive to some of the ingredients.

### Contraindications:

Use of the product is contraindicated when:  
- the skin is broken skin  
- there is swelling at the application site  
- do not stick on the wound directly  
- the duration of use should not exceed 3-4 hours  
- skin needs to be cleaned before use.  
- the user has known allergies to rubber

### Warnings:

Do not apply too tight.  
Ensure that the skin is cleaned before application.  
Do not use the tape if the packaging is damaged or affected with damp and mildew.  
Consult your Health Care Provider if you experience any reaction.  
Do not store above 25 degrees Celsius. Store in original packaging to protect from light.

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