SELECT PROFCARE

PROFCARE COACH TAPE

Style no. 720077 / 720078 / 720079

Material:

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EN

Intended use:

The PROFCARE Coach tape is intended to be used to support the body parts in sports athletes.
Is intended to prevent injury or reinjury by stabilizing body parts in sports athletes.
To prevent and treat injuries by immobilization
This is a non-elastic tape.
It is for use on the skin only.
It is for single, short term use and is a non-sterile device.

Instructions for use:

Fix the top of the tape. Keep a constant traction for the first circle. Overlap the next circle with half width on the first circle. To assure good stickiness, let the final circle fully cover the previous circle. Keep a constant pressure with the palm for a few second on the end of the tape. Skin needs to be cleaned before use. Duration of use should not exceed 3-4 hours

Warnings and contraindications:

Contraindications:

Use of the product is contraindicated when:

- the skin is broken
- there is swelling at the application site
- the user has known allergies to rubber
- the product contains Rosin
- the product contains Latex

Warnings:

Do not apply too tight
Do not store above 25 degrees.
Store in original packaging to protect from sun light
Ensure that the skin is cleaned before application.
Symptoms of overexposure of the acrylic acid may include redness or discolouration, swelling, itching, burning or blistering.
Consult your Health Care Provider if you experience any reaction.

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