SELECT

PROFCARE

PROFCARE K

Style no. 720003

Material:

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EN

Intended use:

The PROFCARE K is intended to be used to support the body parts of sports athletes.
It is for use on the skin only.
It is for single, short-term use and is a non-sterile device.
Heals and prevents sports injuries.

Instructions for use:

Unpack the device and remove the backing tape. Unwind the tape to the correct length and cut or tear across the width of the fabric. Remove the packing paper is removed and apply the tape on the skin.
Skin needs to be cleaned before use.
Duration of use is maximum 3-5 days.

Warnings and contraindications:

Do not use the tape if you have allergies or are sensitive to some of the ingredients.

Contraindications:

Use of the product is contraindicated when:
- the skin is broken
- there is swelling at the application site
- the user has known allergies to rubber

Warnings:

Do not apply too tight
Ensure that the skin is cleaned before application
Do not store above 25 degrees Celsius. Store in original packaging to protect from sun light.
Symptoms of overexposure of the acrylic acid may include redness or discoloration, swelling, itching, burning, or blistering.
Consult your Health Care Provider if you experience any reaction.

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